paper size 5.5" x 3 1/4"

**InVision-Plus® NEUTRAL® Intraluminal  
Protection System**

This product includes the InVision-Plus® NEUTRAL® Intraluminal Protection System. The InVision-Plus® NEUTRAL® is accessed by a standard male-luer syringe or I.V. set connector. It is recommended that a male luer-lock connector be used in the InVision-Plus® NEUTRAL® for a secure connection. The InVision-Plus® NEUTRAL® is bi-directional and luer-locking. InVision-Plus® NEUTRAL® will automatically close when the syringe or male luer I.V. set connector is removed.

**Directions: Applying to vascular access device**

1. Strict aseptic technique should always be used when handling the InVision-Plus® NEUTRAL®.
2. Attach the InVision-Plus® NEUTRAL® to the desired vascular access device by rotating the InVision-Plus® NEUTRAL® in a clockwise direction until secure.

**Directions: Accessing the InVision-Plus® NEUTRAL®**

1. To access the InVision-Plus® NEUTRAL® swab the septum and thread area with a facility approved disinfectant.
2. Position the syringe or I.V. set male-luer connector tip on the InVision-Plus® NEUTRAL® Septum at a 45 degree angle.
3. Push forward, rotate syringe or I.V. set male-luer connector clockwise until secure.
4. To disconnect the syringe or I.V. set male-luer connector, simply rotate counterclockwise. The InVision-Plus® NEUTRAL® will automatically close.
5. The InVision-Plus® NEUTRAL® has only 0.027mL in priming volume.

**NOTES:** Flush InVision-Plus® NEUTRAL® after each use per institutional protocol using normal saline, or normal saline and heparin.

It is recommended that this device be changed per CDC guidelines or per validated facility protocol. The CDC recommends that tubing used to administer lipid emulsions and TNA solutions be changed every 24 hours.

Observe appropriate infection control procedures.

Federal (USA) law restricts this device to sale by or on the order of a physician.

**CAUTION:** The InVision-Plus® NEUTRAL® I.V. Connector may not be compatible with every male-luer connector. Some I.V. sets/syringes utilize a male-luer connector with a lumen opening diameter of 0.063" or less which may cause damage to the InVision-Plus® NEUTRAL® resulting in septum damage, reduction or loss of fluid flow. DO NOT use any male-luer device unless certain that the male-luer connector opening is 0.064" or larger, or written verification of compatibility has been received from RyMed Technologies, Inc. Please CALL (512) 301-1949 for verification.

Do Not use needles with connector. Do Not use luer lock end caps on connector.

InVision-Plus® and NEUTRAL® are registered trademarks of RyMed Technologies, Inc., Franklin, TN 37064  
U.S. Patents: #6,113,068 -#6,299,131-#6,994,315 and other U.S. and foreign patents pending

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InVision-Plus® et NEUTRAL® sont des marques déposées de RyMed Technologies, Inc., Franklin, TN 37064
**Medical Components, Inc.**
Harleysville, PA 19438 U.S.A.