GRIP-LOK®

**EN** Catheter Fixation for MedComp® PICC/CVC

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PN40599CA Rév. 9/21 A

TIDI® Products, LLC  
570 Enterprise Drive  
Neenah, WI 54956 USA  
*Grip-Lok® is a registered trademark of TIDI® Products, LLC.*

*Medcomp® is a trademark of Medical Components, Inc. registered in the United States.*

Distributed By :  
Medical Components, Inc.  
1499 Delp Drive  
Harleysville, PA 19438 USA  
Phone: 215.256.4201  
Fax: 215.256.1787  
www.medcompnet.com

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


**EN**

English

1. **Select** the area for the placement of the Grip-Lok. Note: The catheter should be already inserted in the patient.
2. **Prepare** the skin according to the standard hospital protocol for dressing application. Hair removal may be required on some patients for better adhesion.
3. **Open** the top flap, remove the interior liner and hold the flap open.
4. **Slide** the Grip-Lok under the catheter and adhere in the exposed adhesive cut-out space provided.
5. **Secure** the top flap over the catheter.
6. **Hold** the Grip-Lok in position while removing the paper backing from one side, then the other, to secure on the skin surface.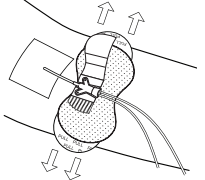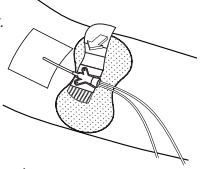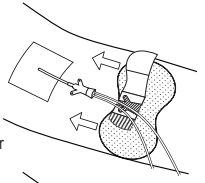

**Direction:** for added skin protection, skin prep may be used prior to application of securement device.

**Note:** replace securement device if soiled or saturated in fluid.

**Note:** use of an alcohol swab will ease removal of securement device from the skin.

### ⚠ Caution

- ⊗ Sterile, Single use: Do not reuse, reprocess or resterilize. Reuse of device creates a potential risk of serious injury and/or infection which may lead to death.

This product and packaging does not contain natural rubber latex or DEHP.

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