- EN** Catheter Fixation for MedComp® PICC/CVC
- DE** Katheter Fixierung für MedComp® PICC/CVC
- FR** Fixation de Cathéter pour MedComp® PICC/CVC
- IT** Fissaggio del catetere MedComp® PICC/CVC
- NL** Catheter Fixatie voor MedComp® PICC/CVC
- SV** Kateter Fixering för MedComp® PICC/CVC
- ES** La Fijación del Catéter para el MedComp® PICC/CVC
- PT** Fixação de catéter para MedComp® PICC/CVC
- EL** Η δέσμευση για MedComp® PICC/CVC
- FI** Katetri Pakkomielle MedComp® PICC/CVC
- DA** Kateter Fiksering for MedComp® PICC/CVC
- IS** Catheter Fixation pentru MedComp® PICC/CVC
- NO** Kateter Fiksering for MedComp® PICC/CVC
- PL** Catheter Utrwalania dla MedComp® PICC/CVC

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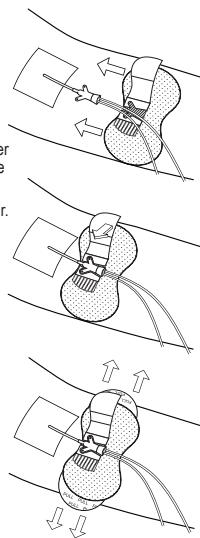
INSTRUCTIONS **ES**

English

- Select the area for the placement of the Grip-Lok. Note: The catheter should be already inserted in the patient.
 - Prepare the skin according to the standard hospital protocol for dressing application. Hair removal may be required on some patients for better adhesion.
 - Open the top flap, remove the interior liner and hold the flap open.
 - Slide the Grip-Lok under the catheter and adhere in the exposed adhesive cut-out space provided.
 - Secure the top flap over the catheter.
 - Hold the Grip-Lok in position while removing the paper backing from one side, then the other, to secure on the skin surface.
- Direction:** for added skin protection, skin prep may be used prior to application of security device.
- Note:** replace security device if soiled or saturated in fluid.
- Note:** use of an alcohol swab will ease removal of security device from the skin.

Caution

(X) Sterile, Single use: Do not reuse, reprocess or resterilize. Reuse of device creates a potential risk of serious injury and/or infection which may lead to death.

This product and packaging does not contain natural rubber latex or DEHP.

(X) Do not use if the package is damaged.

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