

BARBED LUER LOCK ADAPTOR INSTRUCTIONS FOR USE

INTENDED USE:

- Connection point for peritoneal dialysis catheter and male luer.

DESCRIPTION:

- Plastic barbed adaptor with female luer end.





PRECAUTIONS:

- Do not over tighten- may cause cracking of the luer and make removal difficult.
- Do not use hemostats or another instrument to remove cap – may cause cracking.
- Use only luer lock connectors.

POTENTIAL COMPLICATIONS:

- Disconnection
- Infection
- Leakage
- Occlusion

WARNINGS:

- Federal Law (USA) restricts the device to sale by or on the order of a physician.
- Do not re-sterilize by any method. 
- Re-use may lead to infection or illness/injury.
- This device is for Single Use Only. 
- The manufacturer shall not be liable for any damages caused by re-use or re-sterilization of the device.
- Contents sterile and non-pyrogenic in unopened, undamaged package. 
- STERILIZED BY ETHYLENE OXIDE STERILE EO
- Do not use if any sign of product damage is visible. 
- Do not use if package is opened or damaged.

INSTRUCTIONS FOR USE:

- Push barbed end of adaptor so that catheter end is over the barbs.
- Barbed adaptor should fit tightly within catheter end.
- Tug gently to ensure the connection is secure.
- Visually inspect for catheter damage before use.

Medcomp® does not recommend a particular technique for the use of this device. The physician should evaluate their appropriateness according to individual patient conditions and his or her medical training and experience.

WARRANTY









Medcomp® WARRANTS THAT THIS PRODUCT WAS MANUFACTURED ACCORDING TO APPLICABLE STANDARDS AND SPECIFICATIONS. PATIENT CONDITION, CLINICAL TREATMENT, AND PRODUCT MAINTENANCE MAY AFFECT THE PERFORMANCE OF THIS PRODUCT. USE OF THIS PRODUCT SHOULD BE IN ACCORDANCE WITH THE INSTRUCTIONS PROVIDED AND AS DIRECTED BY THE PRESCRIBING PHYSICIAN.

Because of continuing product improvement, prices, specifications, and model availability are subject to change without notice. Medcomp® reserves the right to modify its products or contents without notice.

Medcomp® is a registered trademark of Medical Components, Inc.

 **Medical Components, Inc.**

1499 Delp Drive
Harleysville, PA 19438 U.S.A.
Tel: 215-256-4201
Fax: 215-256-1787
www.medcompnet.com

5.1.1 	Manufacturer *	5.6.3 	Non-pyrogenic *
5.1.4 	Use-by Date *	5.3.2 	Keep Away from Sunlight *
5.4.2 	Do Not Re-use *	5.2.6 	Do Not Resterilize *
5.3.4 	Keep Dry *	Rx Only	Prescription Use Only ***
5.1.5 LOT	Batch/Lot Number *	5.1.6 REF	Catalogue Number *
5.2.8 	Do Not Use if Package is Damaged *		
5.2.3 STERILE EO	Sterilized Using Ethylene Oxide *		

* This symbol is in accordance with ISO 15223-1.

*** FDA guidance Use of Symbols in Labeling.