

**LINE CLAMP  
INSTRUCTIONS FOR USE**

**INTENDED USE:**

- Used to temporarily occlude catheter when not in use.

**DESCRIPTION:**

- Molded plastic clamp




**POTENTIAL COMPLICATIONS:**

- Catheter damage
- Bleeding
- Fluid leakage
- Infection

**PRECAUTIONS:**

- Clamping of the tubing repeatedly in the same location may weaken tubing.
- Failure to fully engage clamp may result in leakage.

**WARNINGS:**

- Federal Law (USA) restricts the device to sale by or on the order of a physician.
- This device is for Single Use Only. 
- Do not resterilize by any method. 
- Re-Use may lead to infection or illness/injury.
- The manufacturer shall not be liable for any damages caused by re-use or resterilization of the device.
- Contents sterile and non-pyrogenic in unopened, undamaged package.
- STERILIZED BY ETHYLENE OXIDE STERILE EO
- Do not use if package is opened or damaged. 
- Do not use if any sign of product damage is visible.

**INSTRUCTIONS FOR USE:**

- Place clamp on exposed portion of catheter.
- Engage clamp to close.
- Open clamp to allow access.
- Discard biohazard according to facility protocol.


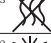





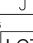
Medcomp® does not recommend a particular technique for the use of this device. The physician should evaluate their appropriateness according to individual patient conditions and his or her medical training and experience.

**WARRANTY**

**Medcomp® WARRANTS THAT THIS PRODUCT WAS MANUFACTURED ACCORDING TO APPLICABLE STANDARDS AND SPECIFICATIONS. PATIENT CONDITION, CLINICAL TREATMENT, AND PRODUCT MAINTENANCE MAY AFFECT THE PERFORMANCE OF THIS PRODUCT. USE OF THIS PRODUCT SHOULD BE IN ACCORDANCE WITH THE INSTRUCTIONS PROVIDED AND AS DIRECTED BY THE PRESCRIBING PHYSICIAN.**

Because of continuing product improvement, prices, specifications, and model availability are subject to change without notice. Medcomp® reserves the right to modify its products or contents without notice.

*Medcomp® is a registered trademark of Medical Components, Inc.*

5.1.1 	Manufacturer *	5.6.3 	Non-pyrogenic *
5.1.4 	Use-by Date *	5.3.2 	Keep Away from Sunlight *
5.4.2 	Do Not Re-use *	5.2.6 	Do Not Resterilize *
5.3.4 	Keep Dry *	Rx Only	Prescription Use Only ***
5.1.5 <span style="border: 1px solid black; padding: 2px;">LOT</span>	Batch/Lot Number *	5.1.6 REF	Catalogue Number *
5.2.8 	Do Not Use if Package is Damaged *		
<span style="border: 1px solid black; padding: 2px;">EC REP</span>	Authorized Representative in the European Community *		
5.2.3 <span style="border: 1px solid black; padding: 2px;">STERILE EO</span>	Sterilized Using Ethylene Oxide *		

\* This symbol is in accordance with ISO 15223-1.  
\*\*\* FDA guidance Use of Symbols in Labeling.



MPS Medical Product Service GmbH  
Borngasse 20  
35619 Braunfels  
Germany



 **Medical Components, Inc.**

1499 Delp Drive  
Harleysville, PA 19438 U.S.A.  
Tel: 215-256-4201  
Fax: 215-256-1787  
www.medcompnet.com