

VEIN PICK INSTRUCTIONS FOR USE

INTENDED USE:

- Instrument designed to facilitate catheter placement into vessel.





DESCRIPTION:

- Plastic instrument with grooved feature to facilitate catheter placement.

POTENTIAL COMPLICATIONS:


- Bleeding
- Hematoma
- Infection
- Laceration of vessel
- Perforation of vessel
- Tissue damage

WARNINGS:

- Federal Law (USA) restricts the device to sale by or on the order of a physician.
- This device is for Single Use Only. 
- Do not resterilize by any method. 
- Re-use may lead to infection or illness/injury.
- The manufacturer shall not be liable for any damages caused by re-use or reesterilization of the device.
- Contents sterile and non-pyrogenic in unopened, undamaged package.
- STERILIZED BY ETHYLENE OXIDE STERILE EO
- Do not use if package is opened or damaged. 
- Do not use if any sign of product damage is visible. 
- Caution when using this device. Be aware of sharps.

INSTRUCTIONS FOR USE:

- Perform vessel incision after vessel is isolated and stabilized to prevent bleeding and air aspiration.
- Insert the tapered end through the incision and advance it into the vessel.
- Slide the catheter tip into the grooved underside of the pick.

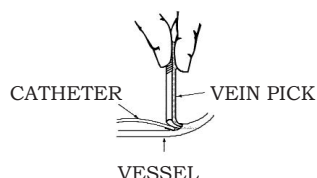

- Advance the catheter tip into the vessel.
- Withdraw the vein pick.
- Continue catheter insertion procedure.
- Device should be disposed of in accordance with policies and procedures for contaminated products.


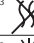






Medcomp® does not recommend a particular technique for the use of this device. The physician should evaluate the appropriateness of the device according to individual patient conditions and his or her medical training and experience.

WARRANTY

Medcomp® WARRANTS THAT THIS PRODUCT WAS MANUFACTURED ACCORDING TO APPLICABLE STANDARDS AND SPECIFICATIONS. PATIENT CONDITION, CLINICAL TREATMENT, AND PRODUCT MAINTENANCE MAY AFFECT THE PERFORMANCE OF THIS PRODUCT. USE OF THIS PRODUCT SHOULD BE IN ACCORDANCE WITH THE INSTRUCTIONS PROVIDED AND AS DIRECTED BY THE PRESCRIBING PHYSICIAN.

Because of continuing product improvement, prices, specifications, and model availability are subject to change without notice. Medcomp® reserves the right to modify its products or contents without notice.

Medcomp® is a registered trademark of Medical Components, Inc.

5.1.1 	Manufacturer *	5.6.3 	Non-pyrogenic *
5.1.4 	Use-by Date *	5.3.2 	Keep Away from Sunlight *
5.4.2 	Do Not Re-use *	5.2.6 	Do Not Resterilize *
5.3.4 	Keep Dry *	Rx Only	Prescription Use Only ***
5.1.5 LOT	Batch/Lot Number *	5.1.6 REF	Catalogue Number *
5.2.8 	Do Not Use if Package is Damaged *		
5.2.3 STERILE EO	Sterilized Using Ethylene Oxide *		

* This symbol is in accordance with ISO 15223-1.

*** FDA guidance Use of Symbols in Labeling.

 Medical Components, Inc.
1499 Delp Drive
Harleysville, PA 19438 U.S.A.
Tel: 215-256-4201
Fax: 215-256-1787
www.medcompnet.com