



## SAPHENOUS VEIN CANNULA

**DESCRIPTION:** The saphenous vein cannulae are devices used to provide access to the saphenous vein.

**INDICATIONS FOR USE:** The saphenous vein cannulae are intended to provide access to the saphenous vein.

**CONTRAINDICATION:** This device is not designed, sold, or intended for use except as indicated.

**WARNINGS:**

- Read instructions carefully. The attending clinician is solely responsible for the set-up and use of this device.
- Observe aseptic technique with all tubing connections.
- Do not over-tighten connections.
- Federal Law (USA) restricts the device to sale by or on the order of a physician.
- Inspect for damage before use. Sterility and/or product performance may be compromised.
- Do not use if package is opened or damaged.
- Do not use if any sign of product damage is visible.
- This device is for Single Use Only.
- Do not re-sterilize by any method.
- Re-Use may lead to infection or illness/injury.
- The manufacturer shall not be liable for any damages caused by re-use or re-sterilization of the device.
- Contents sterile and non-pyrogenic in unopened, undamaged package. STERILIZED BY ETHYLENE OXIDE
- Discard biohazard according to facility protocol.

**CAUTIONS:**

- Observe all additional warnings and/or cautions contained in the Directions for Use below.

**DIRECTIONS FOR USE:**

1. Flush all air from the cannula.
2. Place the cannula tip into the vessel to be accessed.

**Caution:** To prevent vessel damage, monitor delivery pressure during infusions.

**WARRANTY**

**Medcomp® WARRANTS THAT THIS PRODUCT WAS MANUFACTURED ACCORDING TO APPLICABLE STANDARDS AND SPECIFICATIONS. PATIENT CONDITION, CLINICAL TREATMENT, AND PRODUCT MAINTENANCE MAY AFFECT THE PERFORMANCE OF THIS PRODUCT. USE OF THIS PRODUCT SHOULD BE IN ACCORDANCE WITH THE INSTRUCTIONS PROVIDED AND AS DIRECTED BY THE PRESCRIBING PHYSICIAN.**

Because of continuing product improvement, prices, specifications, and model availability are subject to change without notice. Medcomp® reserves the right to modify its products or contents without notice.

5.1.1 	Manufacturer *	5.6.3 	Non-pyrogenic *
5.1.4 	Use-by Date *	5.3.2 	Keep Away from Sunlight *
5.4.2 	Do Not Re-use *	5.2.6 	Do Not Resterilize *
5.3.4 	Keep Dry *	Rx Only	Prescription Use Only ***
5.1.5 	Batch/Lot Number *	5.1.6 REF	Catalogue Number *
5.2.8 	Do Not Use if Package is Damaged *		
5.2.3 	Sterilized Using Ethylene Oxide *		

\* This symbol is in accordance with ISO 15223-1.

\*\*\* FDA guidance Use of Symbols in Labeling.

**Medical Components, Inc.**  
 1499 Delp Drive  
 Harleysville, PA 19438 U.S.A.  
 Tel: 215-256-4201  
 Fax: 215-256-1787  
 www.medcompnet.com