

paper size 5.5" x 3 1/4"






Captive® J-Straightener Use Instructions

To release guidewire from dispenser:

1. Grasp the shoulder of the J-Straightener between the forefinger and thumb.
2. Gently pull on the J-Straightener shoulder until it is just removed from the dispenser tubing.
3. Holding the guidewire and the shank of the J-Straightener, fully withdraw the guidewire from the dispenser.

CAUTION: DO NOT grasp and pull the guidewire prior to releasing the J-Straightener. Damage to the guidewire may occur if it is pulled against the restraint of the J-Straightener.

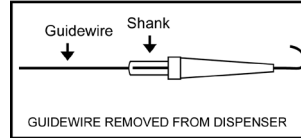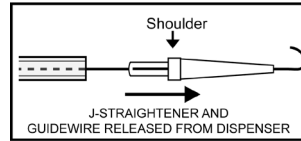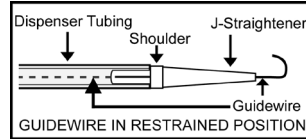

REV. 8/21 A

Captive® J-Straightener is a registered trademark of Lake Region Manufacturing, Inc.

PN 4191CA






Captive® J-Straightener - mode d'emploi

Pour libérer le fil-guide du distributeur :

1. Saisir le lobe du J-Straightener entre l'index et le pouce.
2. Tirer doucement sur le lobe du J-Straightener jusqu'à ce qu'il soit retiré de la tubulure du distributeur.
3. En tenant le fil-guide et la tige du J-Straightener, retirer complètement le fil-guide du distributeur.

MISE EN GARDE : NE PAS saisir et tirer sur le fil-guide avant de relâcher le J-Straightener : Le fil-guide peut être endommagé s'il est tiré contre la retenue du J-Straightener.

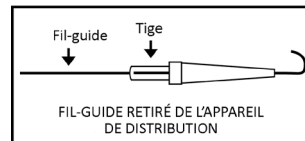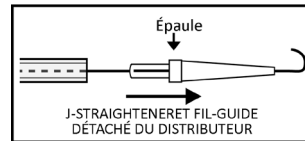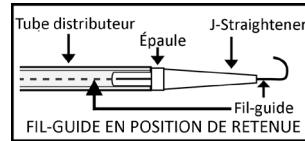

Rév. 8/21 A

Captive® J-Straightener est une marque déposée de Lake Region Manufacturing inc.

PN 4191CA