



# CAPTIVE® J-STRAIGHTENER INSTRUCTIONS FOR USE

## TO RELEASE GUIDEWIRE FROM DISPENSER:


1. Grasp the shoulder of the J-Straightener between the forefinger and thumb.

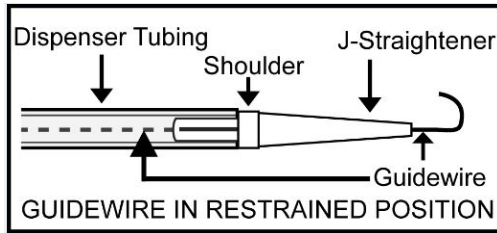

2. Gently pull on the J-Straightener shoulder until it is just removed from the dispenser tubing.

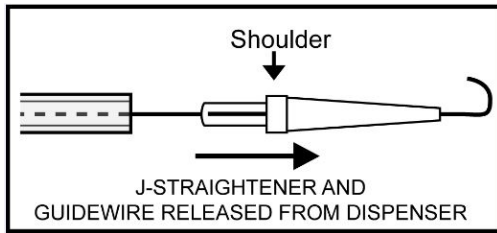

3. Holding the guidewire and the shank of the J-Straightener, fully withdraw the guidewire from the dispenser.

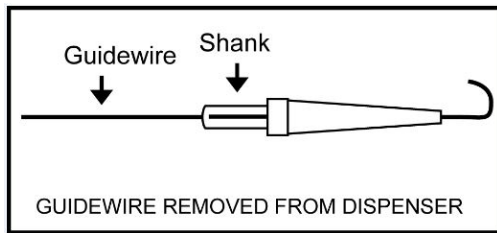

**Note:** Discard biohazard according to facility protocol.

**CAUTION: DO NOT grasp and pull the guidewire prior to releasing the J-Straightener. Damage to the guidewire may occur if it is pulled against the restraint of the J-Straightener.**

### WARRANTY

**Medcomp® WARRANTS THAT THIS PRODUCT WAS MANUFACTURED ACCORDING TO APPLICABLE STANDARDS AND SPECIFICATIONS. PATIENT CONDITION, CLINICAL TREATMENT, AND PRODUCT MAINTENANCE MAY AFFECT THE PERFORMANCE OF THIS PRODUCT. USE OF THIS PRODUCT SHOULD BE IN ACCORDANCE WITH THE INSTRUCTIONS PROVIDED AND AS DIRECTED BY THE PRESCRIBING PHYSICIAN.**

Because of continuing product improvement, prices, specifications, and model availability are subject to change without notice. Medcomp® reserves the right to modify its products or contents without notice.

*Medcomp® is a registered trademark of Medical Components, Inc.*

*Captive® J-Straightener is a registered trademark of Lake Region Manufacturing, Inc.*

**EC REP**

MPS Medical Product Service GmbH  
Borngasse 20  
35619 Braunfels  
Germany

**CE**  
**2797**

5.1.1	Manufacturer *	5.6.3	Non-pyrogenic *
5.1.4	Use-by Date *	5.3.2	Keep Away from Sunlight *
5.4.2	Do Not Re-use *	5.2.6	Do Not Resterilize *
5.3.4	Keep Dry *	Rx Only	Prescription Use Only ***
5.1.5	Batch/Lot Number *	5.1.6	REF Catalogue Number *
5.2.8	Do Not Use if Package is Damaged *		
5.1.2	Authorized Representative in the European Community *		
5.2.3	Sterilized Using Ethylene Oxide *		
5.4.4	Caution, consult Accompanying Documents *		

\* This symbol is in accordance with ISO 15223-1.

\*\*\* FDA guidance Use of Symbols in Labeling.

**Medical Components, Inc.**

1499 Delp Drive  
Harleysville, PA 19438 U.S.A.  
Tel: 215-256-4201  
Fax: 215-256-1787  
www.medcompnet.com