



# Catheter Securement Device

## INSTRUCTION FOR USE

CAUTION: Federal law (U.S.A.) restricts this device to sale by or on the order of a physician.

**Catheter securement device** is intended for single use. Sterility and performance of these devices are guaranteed by the manufacture only if they are intact and prepared as recommended for single use only. The product labeled "for single use only", "disposable" or its equivalent, shall not be reused and should be discarded after their initial use. The reprocessing and reuse of single-use products pose the risk of bacterial growth, contamination, patient injuries and could lead to death. Faulty clean, re-sterilization and tests may allow the transmission of infectious disease between patients; may alter device's mechanical properties and thus risk product failure. It is the end user's responsibility to read and understand the important warnings and to ensure the single use of medical devices. Do not reuse. Catheter securement device is single patient use only and **MUST** be replaced for each patient.

### DESCRIPTION

#### INTENDED USE

Medcomp's Catheter securement device is a sterile device for percutaneous catheters and tubes, consist of a hydrocolloid adhesive with an outer adhesive cover. Hydrocolloid can absorb superfluous exudates and reduce removal trauma. It is conformable and can be easily molded into place to enhance patient comfort and minimize leakage.

#### WARNINGS

1. This device must pay attention to matching medical device size.
2. Prepare the skin is according to the standard hospital protocol for dressing application.
3. Skin prep or hair removal may be required on some patients for better adhesion.
4. After seven(7)days(sooner if the fixation device becomes loose for any reason) remove the old Catheter securement device and replace with a new Catheter securement device.

#### ADVANTAGES

- Breathable and comfortable.
- Maintain reasonable moisture state and absorb exudates.
- Prevents catheter kinking.

Shelf life: 3 years. In situ: 7 days

#### ORDERING INFORMATION - Individually Packaged Sterile

PRODUCT #	CATHETERS SIZE	Units/box
BL0512	5Fr~12Fr	25
BL1218	12Fr~18Fr	25




### PRECAUTIONS FOR USE

- This device is sterilized with ethylene oxide gas and is intended for single use only. Do not re-sterilize and/or reuse this device.
- Do not use if the unit packaging or the product has been damaged or soiled. This is an important precaution before you use the product.
- Use immediately after opening the package and dispose of safely following local procedures for the disposal of medical waste .
- Avoid exposure to water, direct sunlight, extreme temperature or humidity during storage .




## INSTRUCTIONS FOR USE

REF : BL0512 CATHETERS SIZE : 5Fr~12Fr

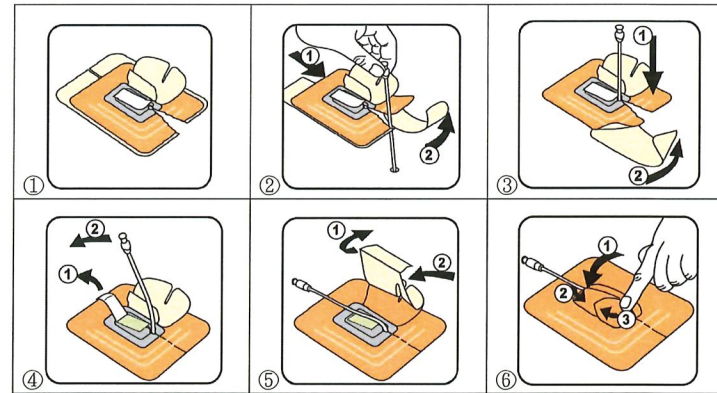

## INSTRUCTIONS FOR USE

REF : BL1218 CATHETERS SIZE : 12Fr~18Fr

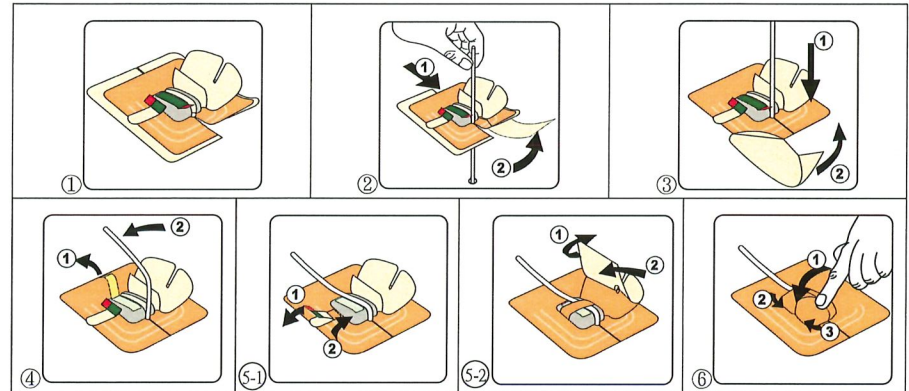

Distributed by Medical Components, Inc. DBA Medcomp®  
 1499 Delp Drive, Harleysville, PA 19438 USA  
 Tel: 215 - 256 - 4201 | Fax: 215 - 256 -1787  
 www.medcompnet.com  
 Medcomp® is a registered trademark of Medical Components, Inc.

