



*med*COMP
C3 WAVE

Hub Bluetooth® Pairing

1. Press and hold the power button for about 8 seconds. The blue power LED will flash rapidly to indicate that the hub has erased pairing information.
2. On the new iPad® go to settings & support and press the “pair” icon under Hub Serial #
3. On the hub press and hold the power button for two seconds until the blue LED begins to flash slowly.
4. On the iPad® press the area marked “Chest Hub”. This will pair the hub with the iPad®.
5. You can verify that the hub has paired correctly by checking that the serial numbers match.



iPad® and Apple® are registered trademarks of Apple Inc.
The Bluetooth® word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks by Medcomp® is under license.
Other trademarks and trade names are those of their respective owners.