



medCOMP
C3 WAVE

Remote Bluetooth® Pairing

1. Press and hold all four arrow buttons for about 5 seconds. This will remove all pairing information from the remote.
2. On the new iPad® go to settings & support and press the “Pair” icon under Remote Serial #
3. On the remote press any button and the blue remote LEDs will begin to flash slowly.
4. On the iPad® press the area marked “Remote”. This will pair the remote with the iPad®.
5. You can verify that the remote has paired correctly by checking that the serial numbers match.

CE2797



MPS Medical Product Service GmbH
Borngasse 20
35619 Braunfels
Germany



iPad® and Apple® are registered trademarks of Apple Inc.
The Bluetooth® word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks by Medcomp® is under license.
Other trademarks and trade names are those of their respective owners.