



12F Tri-Flow
Triple Lumen Catheter

Patient Identification Card

Patient's Name

Lot #

French Size

Insertion Date

Physician's Name

Phone Number

SITE CARE:

- Clean skin around catheter. Cover the exit site with occlusive dressing and leave extensions, clamps, and caps exposed for access by staff.
- Wound dressings must be kept clean and dry.

Catheter is compatible with:

- Bactroban ointment, 2% Mupirocin, polyethylene glycol ointment, N.F.
- Silvadene cream, 1% silver sulfadiazine
- 10% Povidone-Iodine Ointment
- Polysporin or triple antibiotic ointment
- 0.1% Gentamycin
- Hydrogen Peroxide 3% Solution
- 10% Iodophor, Iodine
- ChloroPrep, 2% Chlorhexidine d-Digluconate + 70% Isopropanol (Isopropyl Alcohol)
- Anasept, 0.057% Sodium Hypochlorite
- 70% Alcohol

CATHETER MAINTENANCE: Flush and lock catheter according to your institutional policy.

CAUTION: Patients must not swim, shower, or soak dressing while bathing.

- If profuse perspiration or accidental wetting compromises adhesion of dressing, the medical or nursing staff must change the dressing under sterile conditions.

Consult IFU

